

# Tissue-Tek<sup>®</sup> Glas<sup>™</sup> Coverslips

For use Tissue-Tek<sup>®</sup> Glas<sup>™</sup> Coverslippers

24×50 mm

REF 6416

## 【Intended Purpose】

This product is used to coverslip a glass slide containing an attached section of formalin-fixed and paraffin-embedded (FFPE) specimens or cytopathological specimens in order to protect the specimen from mechanical force or environmental exposure and observe the slide under light microscope for microscopic examination. This product is used with a mounting medium which is placed over the specimen for the attachment of the cover glass to the slide.

## 【Intended User】

Person who is working in the field of pathology, anatomy or clinical cytology with basic knowledge of their respective field (e.g. pathologist, medical laboratory technician) and who is responsible for microscope slide preparation.

## 【Materials required but not supplied】

Microscope Slides, Mounting Medium

## 【How to use】

- Remove plastic cover from Holder.
- Remove foam.
- Place the Holder onto cover glass loading position.
- Slide notch into proper slot.
- Place Holder flat on cover glass loading.

## 【Precautions for use】

- Wear gloves to prevent cuts caused by edges of the glass.
- If any dusty or scattered conditions caused by breakage or disposal occur, wear a dustproof mask (government test-certified) to prevent inhalation, protective eyewear, and protective gloves.
- Avoid touching with wet hands as the glasses will stick to each other.
- Avoid contact of the holder with organic solvents as they melt away holder.
- For the glass installation, refer to the operation manual of the device to be used together with this product.

## 【Safety measures】

- Entry to eye(s): If any dust enters the eye, rinse immediately with running water and consult a physician.

- Adherence to skin: Carefully wipe off without damaging the skin. Consult a physician if any injury or rash has occurred.
- Ingestion: Remove any residue in the mouth, and promptly consult a physician.
- Inhalation: If any dust is inhaled, gargle. If any itching, pain, or discomfort persists in the nose or throat, consult a physician.

## 【Operation and storage conditions】

- Avoid direct sunlight and high-temperature/high-humidity sites. Store indoors in a place with satisfactory natural ventilation.
- Once the holder set in the device is removed from the device, store it with the original cover.

## 【Expiration date】

- The expiration date is printed on the package.

## 【Disposal】

- Disposal to a waste company or dispose of them in accordance with applicable local regulations.

## 【Precautions】

The below is a pictogram as silica dust when glass dust is generated.



- May cause cancer.
- Potential respiratory system or kidney damage may result from inhalation of fine-particulate dust over long periods in large amounts.

## 【Specifications】

200/holder; 5/ case

## 【Note】

If there are any serious incidents that has occurred in relation to the device shall be reported to the manufacturer and your competent authority.

This is a single use device so this is not intended for reuse.

	Sakura Finetek Japan Co., Ltd., 31-1, Nihonbashi-Hamacho 2-chome, Chuo-ku, Tokyo 103-0007 Japan
EC REP	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands
CH REP	MedNet SWISS GmbH Bäderstrasse 18, 5400 Baden, Switzerland
	Sakura Finetek Europe B.V. Flemingweg 10a 2408 AV Alphen aan den Rijn The Netherlands Sysmex Suisse AG Todistrasse 50, 8810 Horgen, Switzerland
Made in Japan	



Please visit our website

Sakura Finetek Japan Co., Ltd. [www.sakura-finetek.com](http://www.sakura-finetek.com)  
Sakura Finetek Europe B.V. [www.sakura.eu](http://www.sakura.eu)

